# CareMat

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# CareMat

## with replaceable cable

The pressure-sensitive bed mat for dementia patients and persons at risk of falling

### Translation of the original instructions

## 1 Safety instructions

- Read these operating instructions thoroughly before putting the device into operation and keep them for future reference.
- Not a safety component in accordance with the EU Machinery Directive must not be used for personal protection or EMERGENCY STOP function.
- Do not use this product other than for its specified application.
- Failure to follow these safety precautions may cause damage to device or objects, serious personal injury, or death.
- Observe all applicable local, national, and international safety standards, codes, and laws.
- After accessing the inside of the device, ensure the cover/protection seal is closed tightly to achieve designated protection rating.
- Warranty claims will be invalid in the event of any damage to the product, to goods or persons that is due to failure to observe these instructions.

### Instructions for nursing staff

- The CareMat operates purely as a support for the nursing staff, but is not a substitute for them.
- The functionality of the CareMat is to be checked each time that it is used.

### Notes for the installer


- The electrical installation work and commissioning of the product may only be carried out by a qualified technician.
- Avoid touching any electronic components.
- ESD safety rules should be observed.
- The product must not be connected to mains voltage (see "Different Types of Electrical Connections").

## 2 Product description


### 2.2 Areas of application and function

The CareMat was primarily designed for use in retirement and nursing homes, hospitals and psychiatric wards as well as for sheltered residential accommodation. The necessary care and supervision of patients with dementia and of those who are at risk of falling represents a significant increase in care expenditure for care practitioners. The CareMat is the ideal support: Risks of falling and the dangers which arise from wandering or loss of orientation can be significantly reduced.


The CareMat, as a pressure-sensitive sensor, is installed directly in front of the carepatient's or sick person's bed and is connected to the internal call system. As soon as the person leaves the bed and steps onto the CareMat this switches on and sends out an alarm signal to the call system.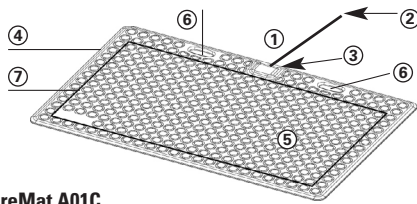

**CareMat A01C**  
**CareMat A11C**  
1100 x 700 mm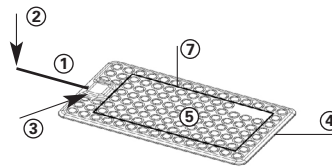

**CareMat B01C**  
**CareMat B11C**  
700 x 400 mm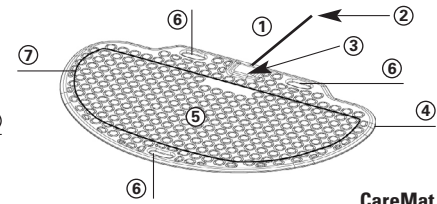


**CareMat C01C**  
**CareMat C11C**  
1100 x 700 mm,  
semi-circular


- ① Replaceable cable for connecting to the call system
- ② Open cable end
- ③ Housing with terminals or universal circuit-board
- ④ Chamfered, wheelchair-accessible edges

- ⑤ Non-slip surface with stud texture design
- ⑥ Carrying handle
- ⑦ Active switching zone


## 3 Day to day operation

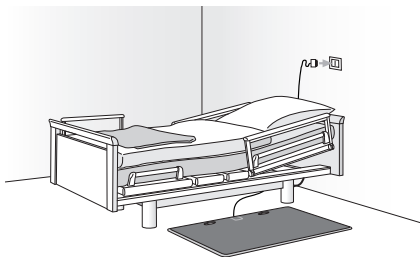Position the CareMat centrally in front of the bed, that avoiding the mat is impossible. Route the cable under the bed in order to avoid the risk of tripping over it. Insert the plug into the correct plug socket device.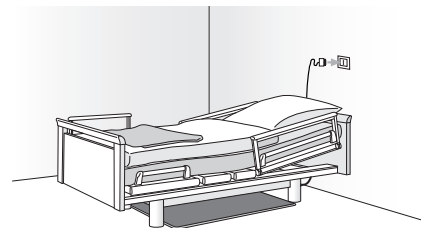

If the CareMat should not be needed, it can be pushed beneath the bed without the cable connection to the call system having to be disconnected.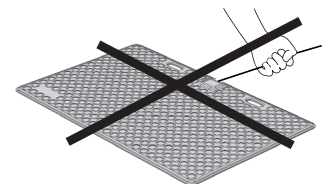

Remove CareMat before moving the bed. This will avoid any unnecessary damage to the mat or connecting cable.

## 4 Cleaning and disinfecting


- Disconnect the connection to the call system prior to any cleaning.
- CareMat can be cleaned using commercially available disinfecting and cleaning agents with max. 70 vol% alcohol. No corrosive agents should be used.
- CareMat is neither autoclavable nor can it be disinfected using steam.

## 5 Transport and storage

- During transport and storage any significant bending of the CareMat should be avoided.
- Store the CareMat in a dry, clean and flat location.
- Do not place any heavy loads on the CareMat and do not stack more than 20 packed mats on top of each other.

## 6 Electrical connections with terminals circuit-board

The CareMat xx1C (C = cable) includes a terminal circuit-board in the housing.

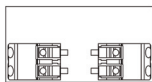The correct plug for connecting to the alarm system and its terminal assignment are to be obtained from the relative system manufacturer.

## 7 Maintenance and fault finding

The CareMat is maintenance-free. If the cable should however be pulled out. It can simply be replaced: open housing, replace and connect up cable. The cable ties around the connecting cable must be stripped using a cable tie tool to avoid any slipping. Close the cover and lightly tighten the screws.

If despite this, the function is not being provided, the following checks should be made:

- Is the call system operating and is the CareMat correctly connected?
  - Is the plug, the plug connection or the cable damaged?
- If the points mentioned above have been checked out, but the functions of the CareMat are, despite this, still not being provided, you should consult a qualified expert.

## 8 Technical specifications

<b>Article</b>	CareMat Ax1C / Bx1C / Cx1C	<b>Cable</b>	Length 3 m, diameter 5.2 ± 0.2 mm
<b>Dimensions (LxWxH)</b>	1100x700x9 mm / 700x400x9 mm / 1100x700x9 mm, height with housing 15 mm	<b>Type of cable</b>	4 pole, LiYYv 4 x 0.34 mm <sup>2</sup>
<b>Weight [kg]</b>	8.2 kg / 3.2 kg / 6.5 kg	<b>Signal output</b>	Mechanical NO contact, transistor NC contact, resistor wiring
<b>Material</b>	Polyurethane (PU)	<b>Electrical load capacity</b>	max. 25 mA at 3 – 24 V DC
<b>Colour</b>	Mat: Careyellow, Caregrey; Cable: white	<b>Type of protection</b>	IP54 without cable end / connecting plug
<b>Surface</b>	Stud texture design	<b>Ambient temperature</b>	0 up to +55 °C
<b>Weight of persons</b>	At least 10 kg		
<b>Chemical resistance</b>	Water (room temperature), body fluids, disinfectant, max. 70 vol% alcohol	<i>Subject to technical modifications</i>	

## 9 EU Declaration of Conformity

 See attachment

## 10 WEEE

Devices with this symbol must be treated separately during disposal. This must be done in accordance with the laws of the respective countries for environmentally sound disposal, processing and recycling of electrical and electronic equipment.

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BBC Bircher Smart Access, BBC Bircher AG, Wiesengasse 20, CH-8222 Beringen, [www.bircher.com](http://www.bircher.com)